


Easy Coordination With Hospital Systems

Data Management System

EFS-1000

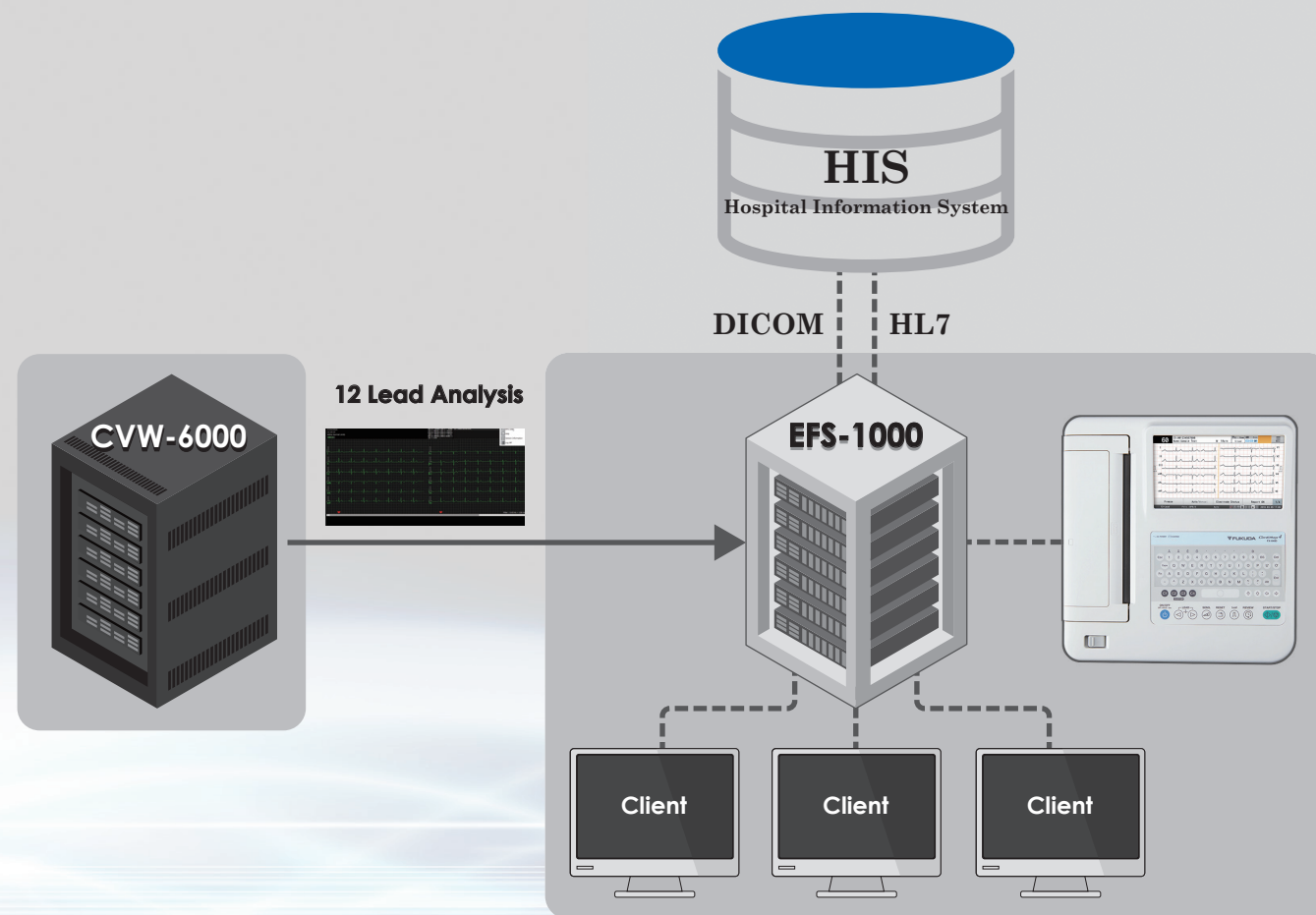

Coordination with the ECG and HIS

The EFS-1000 facilitates seamless ECG-HIS data exchange, efficiently transferring ECG data, orders, and diagnostic results to patient records, enhancing clinical workflow and patient care.

Coordination with EMR

You can receive order information from the EMR server and send examination results from the EFS-1000 to the EMR server.

Integration with CVW-6000

The attending physician can access the ECG of the patient in the ward by launching the EFS directly from the CVW system. This allows for seamless comparison and verification of pre-admission 12-lead ECGs, ECGs immediately after admission, and current ECGs to monitor vital signs during hospitalization.

A simple, easy-to-use ECG examination data management system


1000

FUKUDA DENSHI reserves the right to change specifications without notice.

An ECG Examination Data Management System With Intuitive Controls


View Examination Results

You can check the examination results in raw waveform data on the EFS-1000.

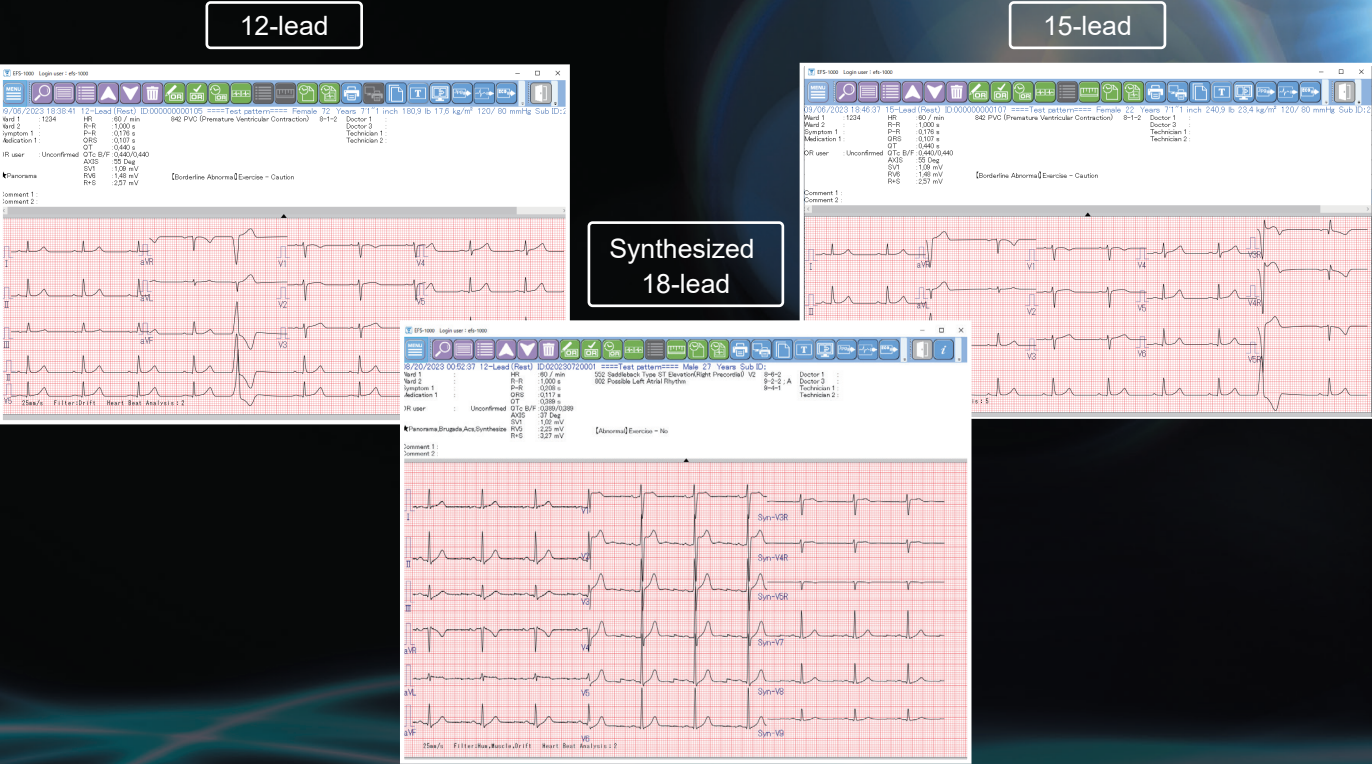

Over-Read

Over-Read refers to the process by which usually a cardiologist would recheck and validate an ECG that was initially interpreted by another doctor or computer algorithm.

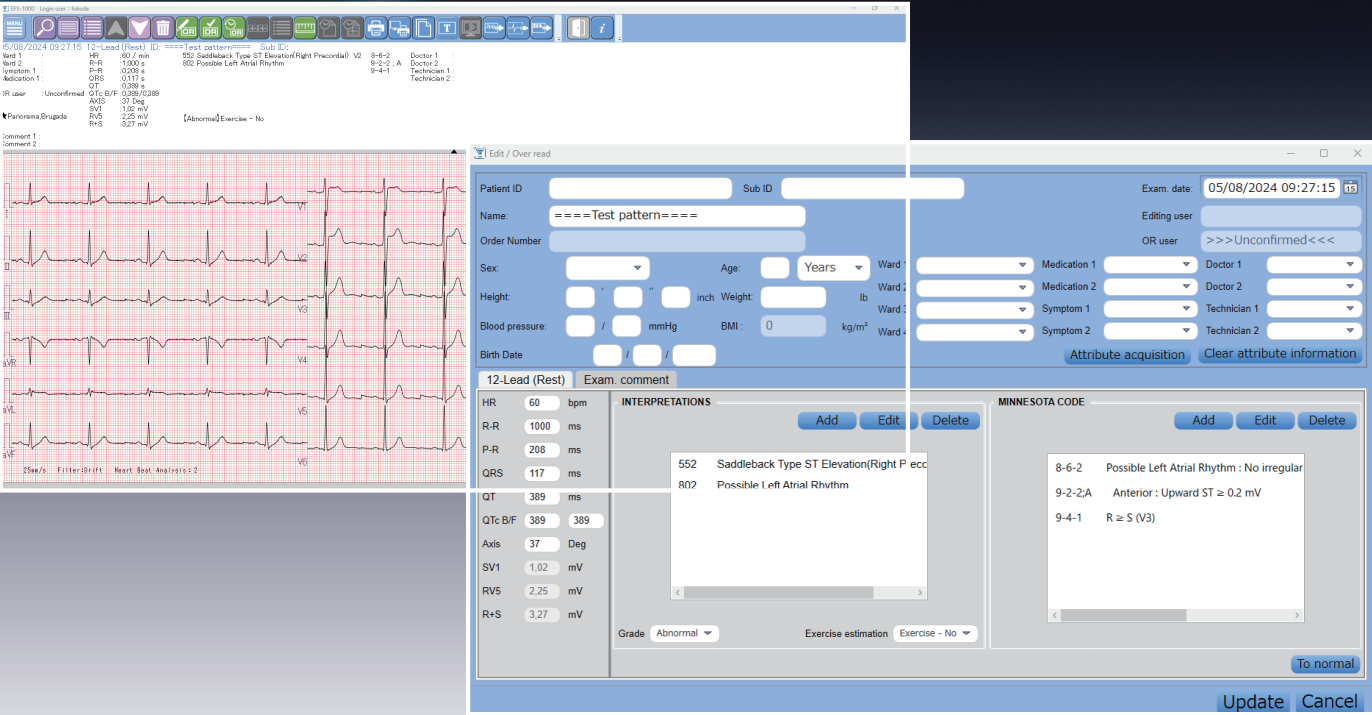

Order Progress Management

Order list Exam. data list

Reservation date Specify manually 08/22/2023 08/30/2023

Search pattern Today

Patient ID Type Select All

Reserve Reception During Exam Suspend Exam. Completed

Auto update Order Details

UrgIn/Outpatient	Reservation date/time	Status	Exam. item	Patient ID	Sub ID	Name	Ag
	08/22/2023 00:00	△	2-Lead (Rest)	000000000009	000000001009	test fukuda	1 Weeks
	08/22/2023 00:00	○	Arrhythmia	000000000005	000000001004		
	08/22/2023 00:00	○	5-Lead FD	000000000008	000000001008	test fukuda	
△ others	08/30/2023 00:00	●	Signal-averaged ECG	000000000003	00000000A022	Patient 03	65 Years
△ Outpatient	08/30/2023 00:00	●	2-lead (Exercise)	000000000005	00000000A022	Patient 05	65 Years
△ Inpatient	08/30/2023 00:00	○	Arrhythmia FD	000000000006	00000000B45	Patient 06	
△ Inpatient	08/30/2023 00:00	△	2-Lead (Rest)	000000000006	00000000B45	Patient 06	
△ Outpatient	08/30/2023 00:00	○	2-Lead (Rest)	000000000007	00000000A0934	Patient 07	77 Years
	08/30/2023 00:00	×	5-Lead FD	000000000008	000000000012	Patient 08	
△ Inpatient	08/30/2023 00:00	○	2-Lead (Rest)	000000000001	0000000000A1	Patient 01	36 Years
△ Outpatient	08/30/2023 00:00	○	Arrhythmia	000000000002	00000000B0342	Patient 02	26 Years

Search results: 11 cases

Order List Detailed Search

Order list Exam. data list

Reservation date Past 6 months 04/12/2023 10/12/2023

Search pattern Today

Patient ID Type Select All

Reserve Reception During Exam Suspend Exam. Completed

Auto update Order Details

Detailed search

Search order

Reservation date Past 6 months 04/12/2023 10/12/2023

Sort Reservation date/time Ascending

None Ascending

Place urgent at top

Patient information

ID Sub ID Name Order number Type ECG Exam.

HIS Exam. item

Status Select All Reserve Reception During Exam Suspend Exit

Exam. information

Ward 1 None Ward 2 None Ward 3 None Ward 4 None Doctor 1 None Doctor 2 None Doctor 3 None Technician 1 None

<Examination information> Search by requesting department, ward, or physician, etc.

<Status> Search Initialize conditions Close

Search by examination progress status